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PL

Nr kat.052

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## SENSIDUO ELECTRIC BREAS PUMP

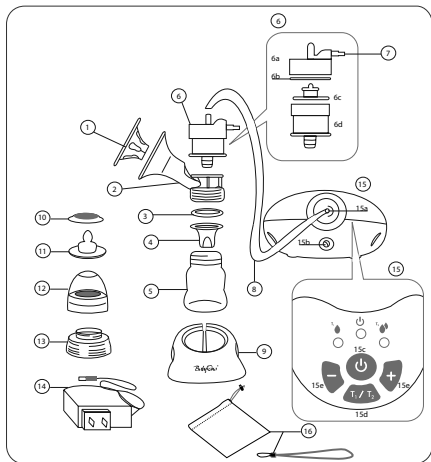
- Two-phase program naturally imitating baby's suckling action
- Adjustable, wide range of suction strength
- Exceptionally quiet operation
- Battery or mains powered

The SENSIDUO electric breast pump has a two-phase program naturally imitating baby's suckling action. The first program (T1) stimulates the breast to lactation with short suckling cycles. The second program (T2) with a longer cycle of operation ensures effective expressing of milk. A wide adjustment range of suction strength of the both programs additionally contributes to efficiency and comfort of the pump operation, adjusting it to individual needs of the mother. The exceptionally quiet operation of the breast pump provides discreet expressing of milk and the possibility of using batteries or mains power pack guarantees its flexible application, also away from home.

## INSTRUCTIONS

Read carefully the instructions for use and warnings before the first use of the BabyOno breast pump. Keep the instructions for future reference!

### BREAST PUMP COMPONENTS

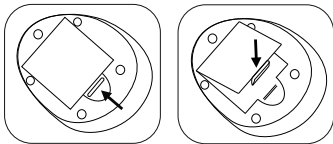
1. silicone massaging pad
2. breast pump funnel
3. seal
4. silicone check valve
5. bottle
6. main body:
  - 6a – top part of the main body
  - 6b – main body seal
  - 6c – main body valve preventing liquids from getting into the pump
  - 6d – bottom part of the main body
7. silicone pipe
8. bottle base
9. little release valve
10. sealing bottle cap
11. teat
12. cap with a ring
13. adaptor for wide neck bottles
14. mains power lead
15. pump:
  - 15a - silicone pipe inlet (7)
  - 15a - mains power lead input (14)
  - 15c – power button
  - 15d – button for changing programs T1 and T2
  - 15e – suction strength increase (+) and reduction (-) button
16. Bag and strap

## INSTRUCTIONS FOR ASSEMBLY:

1. Place the silicone massaging pad (1) on the breast pump funnel (2) and press it to form a whole.
2. Combine the assembled main body (6) with the funnel element of the pump (2) and check whether it is correctly attached.
3. Attach the silicone pipe (7) with one end to the top part of the main body (6a) and with the other end to the silicone pipe inlet (15a) at the back of the pump.
4. Make sure you do not lose the small release valve (9) since in case it is missing the pump will lose the suction function.
5. Attach the seal (3) and silicone check valve (4) to the funnel and next combine the bottle (5) with the breast pump funnel (2). The breast pump is suitable for standard bottle sizes.
6. The use of the adaptor (13) allows combining the breast pump with wide neck bottles.
7. Put the assembled breast pump into the base (8).
8. Connect the mains power lead (14) to the back input of the pump (15b) and plug into an electrical wall socket (DC 3V). Note! Use only an original mains power lead! Other mains power leads may damage the product and may be dangerous in use.

## INSTALLING BATTERIES:

1. Press and remove the battery compartment cover at the bottom of the pump.
2. Insert 4 1.5 V batteries (alkaline batteries required). Pay attention to correct positioning of "+" and "-" battery ends. It is marked inside the battery compartment.
3. Close the battery compartment by pressing the cover.
4. Remove the batteries if the device is not to be used for a longer period of time or if it is mains powered.

## INSTRUCTIONS FOR USE:

1. Remember that stress has a considerable influence and impact on the amount of produced milk. Before you start expressing milk make sure to eliminate any situations stressful to you. Try to relax and always remember about hygiene: wash your hands with soap and water and your breast with water only.
2. Many women feel more thirsty during breastfeeding or expressing of milk – prepare a glass of water or anything else to drink in order not to stop expressing milk.
3. Do not be discouraged with the initial failures – you will shortly become skilled and expressing of milk will not pose a problem to you.
4. Sit down in a position comfortable to you that will enable you to lean slightly forward.
5. Take the previously disinfected breast pump (make sure that all the elements are correctly assembled). Place the breast inside the pump main body adjusting it to the silicone pad in order to place a nipple deep enough to provide its proper stimulation by petals on the pad.
6. Press the power button (15c) for 2 seconds till a corresponding diode on the pump is on.
7. Next choose a proper program for the breast pump operation by pressing the T1/T2 button (15c):

- T1 is a short cycle (sucking in, release, stop) program for massage stimulating lactation.
  - T2 is a longer cycle (sucking in, stop, release) program for proper expressing of milk.
8. The program currently in use is signalled with an illuminated diode on the pump. It is possible to change the T1/T2 mode also during operation of the breast pump.
9. Adjust proper suction strength with + and – button (15e) according to your own needs.
10. Stop expressing your milk when it ceases flowing from the breast.
11. The amount of expressed milk will vary; it depends on whether the lactation process is regulated, on a time of the day and amount of liquids that you drink. Expressing of milk from full breasts may take even 30 minutes.
12. In order to stop operation of the breast pump press the power button (15c) for 2 seconds till a corresponding diode on the pump is off.
13. The use of the breast pump should not be painful. If the process causes pain to you, if you can see redness and irritation of the breast, immediately consult a breastfeeding advisor.
14. Following the completion of expressing of milk wash all the pump elements and prepare them for sterilisation.

## TROUBLESHOOTING

### THE PUMP IS NOT WORKING:

Check that the batteries are correctly positioned in the battery compartment – the battery ends are placed according to the marking on the compartment. Check also that the batteries are not pushed too hard in the compartment.

- If you use the mains power lead make sure that it is correctly connected to the breast pump and plugged into the socket.

### LACK OR WEAKENING SUCTION:

- Make sure that all the breast pump elements are correctly assembled.
- Check that the main body valve (6c) is not stuck to the top part of the main body (6a). For that purpose take the main body into pieces and dry all its elements. Note! Do not remove the main body valve – it prevents liquids from getting into the pump.
- Check that the release valve is present in the top part of the main body. If it is missing use a spare release valve attached to the set.
- Check power of the batteries. If they are low replace them with new ones or connect the mains power lead.
- Check whether there is any air between the breast and the silicone pad. If there is some air there, take out the breast and place it again in the pump funnel.

## ASSISTANCE OF THE BREAST PUMP IN BREASTFEEDING-RELATED PROBLEMS

- **Excess of milk** – when the breast is filled with too much milk it is difficult for the baby to correctly grasp a nipple. By expressing the excess of milk you will make the breast softer and thus you will make it easier for the baby to correctly grasp the nipple.
- **Shortage of milk** – insufficient amount of milk makes the baby more easily irritated and discourages it from sucking the breast. By using the BabyOno breast pump between proper breastfeeding you will stimulate your breast to produce more milk.
- **Nipple shape** – especially in the initial phase of breastfeeding numerous women complain about the shape of their nipples that are not sufficiently protruding which makes it difficult for the baby to correctly grasp the nipple. Regular use of the breast pump will help you to more quickly adapt the breast for feeding.
- **Sore nipples** – before you and your baby become skilled in breastfeeding the nipples may be wounded with gums of the infant who may suck them in an improper way. By using the BabyOno breast pump you give your nipples some time for healing and shortly you will be able to breastfeed the baby again.

## CLEANING OF THE BREAST PUMP

Disassemble all the breast pump parts (in an order opposite to the assembly instructions). Before the first use and after each subsequent use carefully clean all the breast pump elements – EXCEPT FOR the mains power lead (14) and the pump (15)! - in warm water with soap for children and rinse several times. The breast pump elements – EXCEPT FOR the mains power lead (14) and the pump (15)! – may be also washed in the dishwasher (on the top rack only), but you must remember to immediately take out and dry all its elements following the completion of the program.

The mains power lead (14) and the pump (15) should be wiped with a damp cloth only.

After washing the breast pump and each time before its use all the parts having contact with milk should be sterilised:

- silicone massaging pad (1)
- breast pump funnel (2)
- seal (3)
- silicone check valve (4)
- main body (6)
- bottle (5), teat (11), cap with a ring (12), sealing cap (10)
- adaptor for wide neck bottles (13)

In order to do so boil them in water for 5 minutes or sterilise in a microwave or steam steriliser intended for that purpose. Never use for sterilisation any strong chemicals or any other products not intended for that purpose. Do not leave the product in disinfectants for time longer than the recommended one since it may result in its damage and/or impairment.

Store the product in a dry place at room temperature. Do not leave the product in a sun filled or hot place.

The attached teat is suitable for standard BabyOno bottles. Before each use always check by pulling into different directions whether the teat is not damaged, especially when the child already has teeth. Replace a damaged, bitten or misshaped teat with a new one! It is not allowed to make holes in the teat bigger due to the risk of damage to the product and/or child's choking. Due to hygienic reasons and to ensure safety it is recommended to replace a teat every 12 months.

Remember to assemble the sterilised parts of the breast pump and bottle with clean hands.

### For the safety and health of your child!

#### WARNING!

- Always use the product under the supervision of adults. Never leave the child unattended during feeding.
- Never use bottle teats as soothers.
- The continuous and prolonged sucking of liquids may result in dental caries.
- Always check the food temperature before feeding.
- Keep all the breast pump parts away from reach of children.

Product compliant with the EN 14350 standard.

## STORING EXPRESSED MILK

**IMPORTANT:** remember to properly store expressed milk! Improperly stored milk loses its nutritive value and may even do harm to the baby.

We recommend sterile BabyOno bags for storing and freezing breast milk.

STORAGE PLACE	STORAGE TIME	
	FOR HEALTHY BABIES	FOR PREEMIES AND SICK BABIES
At room temperature	8-12 hours	0-1-4 hours
In the fridge ( -5°C)	3-4 days	24-48 hours
In the fridge freezer compartment ( -10°C)	2 weeks	1 week
In the freezer ( -18°C)	6-12 months	3-6 months

#### Notes

- Never refreeze breast milk. Any unused milk should be discarded.
- Do not mix defrosted breast milk with fresh breast milk.
- Only fresh breast milk should be frozen.
- It is recommended to slowly defrost breast milk. For that purpose frozen breast milk should be left in the fridge for reaching a liquid form.
- For making breast milk warm in it recommended to use hot water bath the temperature of which should not exceed 50°C.
- In order to preserve the nutritive value breast milk should not be boiled, warmed up directly on a gas burner or in a microwave!
- The temperature of the milk ready for serving should be the temperature of human body, so not higher than 37°C.

#### WE RECOMMEND ACCESSORIES FOR BREASTFEEDING MOTHERS:

##### 1. Bags for storing and freezing breast milk (No.1039)

Manufactured from special, sterile plastic intended for contact with food and not containing BPA. The attached stickers allow marking the milk and the special construction of the bag allows its comfortable storage with no risk of spilling.

##### 2. Premium lactation pads (No.031,031/60)

Pads with higher absorbency – designed especially for women with abundant lactation to feel comfort and freedom. Light, discreet and ultra thin.

##### 3. Silicone nipple protectors (No.847,848)

Manufactured from super soft and naturally delicate silicone, they protect sensitive and irritated nipples during breastfeeding.

An especially designed construction of the BabyOno Natural, Sensi ,Sensiduo breast pumps enables you to easily change the manual breast pump into an electric one and the other way around, allowing you to adjust the product to your needs.

#### ELECTRIC BREAST PUMPS BABYONO

##### Electric breast pump SENSI (No.051)

- High operational comfort and efficiency
- Adjustable suction strength
- Battery or mains powered

**MANUAL BREAST PUMP NATUREL (No.053)**

- natural mode of operation imitating baby's suckling action
- high operational comfort and efficiency
- easy assembly and cleaning of a small number of elements

An especially designed construction of the BabyOno Natural, Sensi , Sensiduo breast pumps enables you to easily change the manual breast pump into an electric one and the other way around, allowing you to adjust the product to your needs.

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