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EN BREAST MILK STORAGE BAGS

CAPACITY: 100 ml

The packaging contains 20 bags and 20 stickers.

Outer layer: PET (-18°C ~ 120°C)

Inner layer: CPP (-18°C ~ 110°C)

W The bags are suitable for storing and freezing breast milk.

A special material was used for their production that is intended for contact with food. They do not contain bisphenol. Thanks to gamma ray sterilization, the bags ensure the highest hygiene standards. Enclosed stickers make it easier to mark the breast milk storage bag, and the special construction of the bag allows it to be stored conveniently, without a risk of spillage.

The product conforms to the norm PN-EN 14350-1.

Recommended time for milk storage.

	HEALTHY BABIES	PREMATURE AND ILL BABIES
At room temperature	8-12 hours	0-1-4 hours
In a fridge (-5°C)	3-4 days	24-48 hours
In a freezer (-10°C)	2 weeks	1 week
In a deep freezer (-18°C)	6 – 12 months	3-6 months

- Due to the change in composition of the breast milk with the baby's growth, we recommend using the milk within a period of no longer than two weeks.

Breast milk defrosting.

- It is recommended to slowly defrost breast milk. To do so, the frozen breast milk should be placed in the fridge until it is liquid.
- To warm up breast milk it is best to place it in a container with hot water whose temperature should not exceed 50°C.
- To retain its nutritious value, breast milk should not be boiled or heated up directly using a cooker or microwave!
- The temperature of the ready breast milk should have the temperature of a human body, i.e. should not be higher than 37°C.

hot water


boiling


microwave

Caution


- Do not re-freeze already defrosted breast milk. Unused breast milk should be thrown away.
- Do not mix defrosted breast milk with freshly expressed milk.
- Milk storage bags are disposable.
- Maximum hygiene should be kept while using the bags. Remember to wash your hands thoroughly before using them.
- Only fresh milk should be frozen.
- Milk storage bags are meant to be disposable.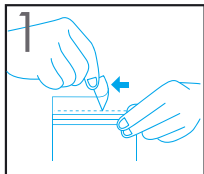


Pull the bag above the click on closure (at the line of cuts).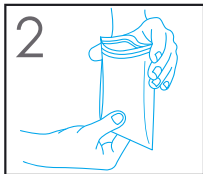


Open the top closure with one hand, while with the other the sides of the bag. Do not put fingers inside and do not blow inside!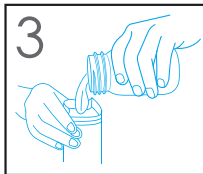

Pour breast milk slowly into the opened bag. Do not exceed the maximum capacity marked with a red line.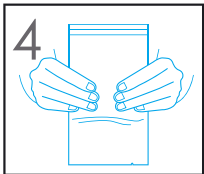

Squeeze the remaining air out slowly and close the click on closure.

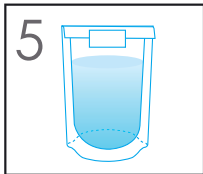Roll the upper part of the bag and seal it with a label. On the label mark the date, hour and quantity of stored milk.

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